

The Hip Huggie™

- Intended for suspected pelvic/hip injury stabilization.
- Eases discomfort in transport.
- Cleans easily.
- Color and number coded straps for easy application.
- Smaller and more economical version of the original Hip Hugger.
- Comes with carrying case.

Instructions

1. Secure device around patient's pelvic/hip area with Velcro closures.
2. Buckle Black straps together.
3. buckle Orange straps together.
4. Simultaneously pull the Orange strap #1 and the Black strap #1 .
5. Simultaneously pull the Orange strap #2 and the Black strap #2.



For more information, call 800-438-6677
or visit www.MorrisonMed.com.